

BENZOS PLUS OPIATES LOW WHOLE BLOOD TOXICOLOGY CONTROL

I. INTENDED USE:

Many analytes can be measured in patients by using analytical test methods. The UTAK Benzos Plus Opiates Low Control is for use as a quality control material for measuring the levels of analytes in blood. It is intended for use on a continuous basis so that a statistical evaluation of testing performance can be obtained.

II. SUMMARY AND PRINCIPLES:

Several different techniques are used for evaluating or estimating the variance of results. The three subjects summarized below must be considered with any test method.

1. PREVENTIVE MEASURES:

These measures are usually contained in the design of the test method and include consideration for reagents, equipment, and operator errors. These measures are designed to minimize variance.

2. QUALITY CONTROL MEASURES:

When a quality control sample is analyzed at the same time and in the same manner as a patient specimen, an estimate of variance is obtained for the test method. This estimate of variance can be compared to the acceptable limits of variance of the test method.

3. STATISTICAL ANALYSIS OF PATIENT RESULTS:

As an aid in evaluating overall test results, the past experience of expected results can be compared to the results of any given test run. For example, it would not be expected that all results of a given test run be in an elevated range.

Quality control materials are widely used as a means to aid in the evaluation of test results. The following subjects are to be considered in the use of any control material.

- | | |
|-----------------|----------------------------|
| 1. Multi-Level | NORMAL / ELEVATED |
| 2. Matrix | HUMAN / ANIMAL / CHEMICAL |
| 3. Availability | SUFFICIENT FOR STATISTICS |
| 4. Form | LIQUID / FROZEN / DRIED |
| 5. Variety | DIFFERENT THAN CALIBRATORS |

The UTAK Benzos Plus Opiates Low Control is prepared from normal human materials and will generate data that checks and evaluates the results of a test method over the normal and elevated ranges. The principles of statistics require that the same material be available for comparison for any given time period. Frozen control materials both extend the usable time period and allow larger quantities to be available. Statistical accuracy requires that a test method be defined for variance and be calibrated with a suitable standard. The quality control materials that are used must be of a sufficient variety so that the measurements and the data that are obtained are independent of the calibration standards. By using a variety of materials, the entire test method can be continuously evaluated to ensure reliable results.

III. PRODUCT DESCRIPTION:

The matrix for the UTAK Benzos Plus Opiates Low Control is prepared from normal human whole blood. Every analyte is added and adjusted to the desired concentration range for each lot prepared (Target Value). Quality control before, during, and after the preparation of the control material ensures that each lot is of the same quality.

IV. PRECAUTIONS:

- Although the whole blood have been tested and found negative for HBsAg by RIA and HIV by EIA, the control material should be treated as any other potentially infectious agent.
- For in vitro diagnostic use only.
- For analytical use only.

V. STORAGE AND STABILITY:

- Store fresh frozen control material at or below -10°C (14°F). Stable to expiration date printed on the insert and label.
- Store thawed control material at or below 2-8°C (35-46°F). **See table for thawed stability.**

VI. PROCEDURE:

- Allow fresh frozen control material to thaw at room temperature with cap on.
- Swirl gently 3-4 minutes to ensure a homogeneous mixture.
- Swirl gently each time an aliquot is removed to ensure a homogeneous mixture.
- Assay control material in same manner as patient specimens, following the exact same instructions from the entire test method.
- Record the results obtained on a quality control chart that describes statistical limits for the test method and the particular lot of control material.

VII. LIMITATIONS:

- Control material is for use in quality control programs only; it is not intended for use as a calibration standard.

- Check the lot number on each vial to be sure it corresponds to the lot number printed on the insert.
- Results are dependent upon proper storage and adequate mixing.
- Control material approximates a patient specimen.

VIII. EXPECTED VALUES:

- Laboratories should establish their own statistical values for precision and expected ranges; these values should fall within ±15% of the Target Value.

Lot Number : <u>A3465</u>		Expiration Date : <u>05/16</u>	
Analyte	Target Value	Units	Thawed Stability (days)
Alprazolam	150	ng/mL	25
α-Hydroxyalprazolam	150	ng/mL	25
Buprenorphine	150	ng/mL	25
Chlordiazepoxide	150	ng/mL	30
Clonazepam	150	ng/mL	30
7-Aminoclonazepam	150	ng/mL	25
Codeine	150	ng/mL	30
Dihydrocodeine	150	ng/mL	25
Diazepam	150	ng/mL	30
Nordiazepam	150	ng/mL	25
Estazolam	150	ng/mL	25
Fentanyl	15	ng/mL	25
Acetyl Fentanyl	15	ng/mL	25
Norfentanyl	15	ng/mL	25
Flunitrazepam	150	ng/mL	30
7-Aminoflunitrazepam	150	ng/mL	25
Flurazepam	150	ng/mL	25
Desalkylflurazepam	150	ng/mL	25
Hydrocodone	150	ng/mL	30
Norhydrocodone	150	ng/mL	25
Hydromorphone	150	ng/mL	30
Lorazepam	150	ng/mL	25
Meperidine	150	ng/mL	30
Normeperidine	150	ng/mL	25
Methadone	150	ng/mL	30
Midazolam	150	ng/mL	25
Morphine	150	ng/mL	30
6-Monoacetylmorphine	150	ng/mL	30
Naloxone	150	ng/mL	25
Nitrazepam	150	ng/mL	25
Oxazepam	150	ng/mL	25
Oxycodone	150	ng/mL	25
Noroxycodone	150	ng/mL	25
Oxymorphone	150	ng/mL	25
Propofol	150	ng/mL	25
Sufentanil	15	ng/mL	25
Tapentadol	150	ng/mL	25
Temazepam	150	ng/mL	25
Tramadol	150	ng/mL	25
Triazolam	150	ng/mL	25
α-Hydroxytriazolam	150	ng/mL	25
Zaleplon	150	ng/mL	25
Zolpidem	150	ng/mL	25
Zopiclone	150	ng/mL	25

UTAK LABORATORIES, INC.
25020 AVENUE TIBBITTS
VALENCIA, CA 91355
TEL: (661) 294-3935
FAX: (661) 294-9272
E-MAIL: INQUIRIES@UTAK.COM

PRODUCT NUMBER:
BENZO+ *****
1x2.5ML VIALS, FROZEN

EC AUTHORIZED REPRESENTATIVE
EMERGO EUROPE
MOLENSTRAAT 15
2513 BH, THE HAGUE
THE NETHERLANDS

