

BENZODIAZEPINES PLUS 400 NG/ML

URINE TOXICOLOGY CONTROL

I. INTENDED USE:

Many analytes can be measured in patients by using analytical test methods. The UTAK Benzodiazepines Plus 400 ng/mL Control is for use as a quality control material for measuring the levels of benzodiazepines in urine. It is intended for use on a continuous basis so that a statistical evaluation of testing performance can be obtained.

II. SUMMARY AND PRINCIPLES:

Several different techniques are used for evaluating or estimating the variance of results. The three subjects summarized below must be considered with any test method.

1. PREVENTIVE MEASURES:

These measures are usually contained in the design of the test method and include consideration for reagents, equipment, and operator errors. These measures are designed to minimize variance.

2. QUALITY CONTROL MEASURES:

When a quality control sample is analyzed at the same time and in the same manner as a patient specimen, an estimate of variance is obtained for the test method. This estimate of variance can be compared to the acceptable limits of variance of the test method.

3. STATISTICAL ANALYSIS OF PATIENT RESULTS:

As an aid in evaluating overall test results, the past experience of expected results can be compared to the results of any given test run. For example, it would not be expected that all results of a given test run be in an elevated range.

Quality control materials are widely used as a means to aid in the evaluation of test results. The following subjects are to be considered in the use of any control material.

- | | |
|-----------------|----------------------------|
| 1. Multi-Level | NORMAL / ELEVATED |
| 2. Matrix | HUMAN / ANIMAL / CHEMICAL |
| 3. Availability | SUFFICIENT FOR STATISTICS |
| 4. Form | LIQUID / FROZEN / DRIED |
| 5. Variety | DIFFERENT THAN CALIBRATORS |

The UTAK Benzodiazepines Plus 400 ng/mL Control is prepared from normal human materials and will generate data that checks and evaluates the results of a test method over the normal and elevated ranges. The principles of statistics require that the same material be available for comparison for any given time period. Dried control materials both extend the usable time period and allow larger quantities to be available. Statistical accuracy requires that a test method be defined for variance and be calibrated with a suitable standard. The quality control materials that are used must be of a sufficient variety so that the measurements and the data that are obtained are independent of the calibration standards. By using a variety of materials, the entire test method can be continuously evaluated to ensure reliable results.

III. PRODUCT DESCRIPTION:

The matrix for the UTAK Benzodiazepines Plus 400 ng/mL Control is prepared from normal human urine. Every analyte is added and adjusted to the desired concentration range for each lot prepared (Target Value). Quality control before, during, and after the preparation of the control material ensures that each lot is of the same quality.

IV. PRECAUTIONS:

- Although the urine donors have been tested and found negative for HBsAg by RIA and HIV by EIA, the control material should be treated as any other potentially infectious agent.
- For in vitro diagnostic use only.
- For analytical use only.

V. STORAGE AND STABILITY:

- Store dried control material at 2-8°C (35-46°F). Stable to expiration date printed on the insert and label.
- Store reconstituted control material at 2-8°C (35-46°F). **See table for reconstituted stability.**

VI. PROCEDURE:

- Remove cap from each vial to be used.
- Reconstitute control material by adding **exactly 5.0 mL of distilled water**, using a **5 mL volumetric pipette** or equivalent.
- Replace cap and let sit 10-15 minutes.
- Swirl gently 3-4 minutes to ensure a homogeneous mixture.
- Swirl gently each time an aliquot is removed to ensure a homogeneous mixture.

- Assay control material in same manner as patient specimens, following the exact same instructions from the entire test method.
- Record the results obtained on a quality control chart that describes statistical limits for the test method and the particular lot of control material.

VII. LIMITATIONS:

- Control material is for use in quality control programs only; it is not intended for use as a calibration standard.
- Check the lot number on each vial to be sure it corresponds to the lot number printed on the insert.
- Results are dependent upon proper storage, reconstitution accuracy, and adequate mixing.
- Control material approximates a patient specimen.

VIII. EXPECTED VALUES:

- Listed in the table below are the Target Value and the *Reference Value*; the *Reference Value* is derived from replicate analysis performed by independent laboratory testing.
- The *Reference Value* is determined by High Performance Liquid Chromatography/Tandem Mass Spectrometry (LC/MS-MS).
- Laboratories should establish their own statistical values for precision and expected ranges; these values should fall within $\pm 15\%$ of the Target Value.

Lot Number : <u>A6837</u>		Expiration Date : <u>03/19</u>		
Analyte	Target Value	Reference Value	Units	Reconstituted Stability (days)
Alprazolam	400	450	ng/mL	25
a-Hydroxyalprazolam	400	390	ng/mL	25
Chlordiazepoxide	400	390	ng/mL	25
Norchlordiazepoxide	400	440	ng/mL	25
Clobazam	400	370	ng/mL	25
Clonazepam	400	410	ng/mL	25
7-Aminoclonazepam*	400	400	ng/mL	n/a
Diazepam	400	350	ng/mL	25
Nordiazepam	400	350	ng/mL	25
Estazolam	400	370	ng/mL	25
Flunitrazepam	400	360	ng/mL	25
7-Aminoflunitrazepam*	400	460	ng/mL	n/a
Flurazepam	400	390	ng/mL	25
Desalkylflurazepam	400	380	ng/mL	25
Hydroxyethylflurazepam	400	360	ng/mL	25
Lorazepam	400	400	ng/mL	25
Midazolam	400	460	ng/mL	25
a-Hydroxymidazolam	400	370	ng/mL	25
Nitrazepam	400	390	ng/mL	25
7-Aminonitrazepam*	400	380	ng/mL	n/a
Oxazepam	400	460	ng/mL	25
Temazepam	400	370	ng/mL	25
Triazolam	400	460	ng/mL	25
α -Hydroxytriazolam	400	380	ng/mL	25
Zolpidem	400	450	ng/mL	25

*7-Amino metabolites have been weighed and spiked into this control material but no claim is made for expected values or stability

UTAK's express and implied warranties (including merchantability and fitness) are conditioned on the observance of UTAK's insert directions with respect to the use of UTAK's products.

For technical assistance call: **UTAK Technical Service (800) 235-3442**

MANUFACTURER:

UTAK LABORATORIES, INC.
25020 AVENUE TIBBITTS
VALENCIA, CA 91355
TEL: (661) 294-3935
FAX: (661) 294-9272
E-MAIL: INQUIRIES@UTAK.COM

PRODUCT NUMBER:

12091
DFID: LUTDM
5x5ML VIALS, DRIED

EC AUTHORIZED REPRESENTATIVE

EMERGO EUROPE
MOLENSTRAAT 15
2513 BH, THE HAGUE
THE NETHERLANDS

