

I. INTENDED USE:

Many analytes can be measured in patients by using analytical test methods. The UTAK DAU HC 1 Control is for use as a quality control material for measuring the levels of analytes in urine. It is intended for use on a continuous basis so that a statistical evaluation of testing performance can be obtained.

II. SUMMARY AND PRINCIPLES:

Several different techniques are used for evaluating or estimating the variance of results. The three subjects summarized below must be considered with any test method.

1. PREVENTIVE MEASURES:

These measures are usually contained in the design of the test method and include consideration for reagents, equipment, and operator errors. These measures are designed to minimize variance.

2. QUALITY CONTROL MEASURES:

When a quality control sample is analyzed at the same time and in the same manner as a patient specimen, an estimate of variance is obtained for the test method. This estimate of variance can be compared to the acceptable limits of variance of the test method.

3. STATISTICAL ANALYSIS OF PATIENT RESULTS:

As an aid in evaluating overall test results, the past experience of expected results can be compared to the results of any given test run. For example, it would not be expected that all results of a given test run be in an elevated range.

Quality control materials are widely used as a means to aid in the evaluation of test results. The following subjects are to be considered in the use of any control material.

- | | |
|-----------------|----------------------------|
| 1. Multi-Level | NORMAL / ELEVATED |
| 2. Matrix | HUMAN / ANIMAL / CHEMICAL |
| 3. Availability | SUFFICIENT FOR STATISTICS |
| 4. Form | LIQUID / FROZEN / DRIED |
| 5. Variety | DIFFERENT THAN CALIBRATORS |

The UTAK DAU HC 1 Control is prepared from normal human materials and will generate data that checks and evaluates the results of a test method over the normal and elevated ranges. The principles of statistics require that the same material be available for comparison for any given time period. Dried materials both extend the usable time period and allow larger quantities to be available. Statistical accuracy requires that a test method be defined for variance and be calibrated with a suitable standard. The quality control materials that are used must be of a sufficient variety so that the measurements and the data that are obtained are independent of the calibration standards. By using a variety of materials, the entire test method can be continuously evaluated to ensure reliable results.

III. PRODUCT DESCRIPTION:

The matrix for the UTAK DAU HC 1 Control is prepared from normal human urine. Every analyte is added and adjusted to the desired concentration range for each lot prepared (Target Value). Quality control before, during, and after the preparation of the control material ensures that each lot is of the same quality.

IV. PRECAUTIONS:

1. Although the urine has been tested and found negative for HBsAg by RIA and HIV by EIA, the control material should be treated as any other potentially infectious agent.
2. For in vitro diagnostic use only.
3. For analytical use only.

V. STORAGE AND STABILITY:

1. Store dried control material at 2-8°C (35-46°F). Stable to expiration date printed on the insert and label.
2. Store reconstituted control material at or below 2-8°C (35-46°F). **See table for reconstituted stability.**

VI. PROCEDURE:

1. Remove cap from each vial to be used.
2. Reconstitute control material by adding exactly 10.0 mL of distilled water, using a 10 mL volumetric pipette or equivalent.
3. Replace cap and let sit 10-15 minutes.
4. Swirl gently 3-4 minutes to ensure a homogeneous mixture.
5. Swirl gently each time an aliquot is removed to ensure a homogeneous mixture.
6. Assay control material in same manner as patient specimens, following the exact same instructions from the entire test method.
7. Record the results obtained on a quality control chart that describes statistical limits for the test method and the particular lot of control material.

VII. LIMITATIONS:

1. Control material is for use in quality control programs only; it is not intended for use as a calibration standard.
2. Check the lot number on each vial to be sure it corresponds to the lot number printed on the insert.
3. Control material approximates a patient specimen.

VIII. EXPECTED VALUES:

1. Laboratories should establish their own statistical values for precision and expected ranges.

Lot Number : C4043		Expiration Date : 04/22			
Analyte	Target Value	Verified Value	UTAK Cutoffs	SAMHSA Cutoffs	Units
AMPHETAMINES					
d-Amphetamine	375	356	500	250	ng/mL
d-Methamphetamine	375	341			
MDA	375	370			
MDEA	375	325			
MDMA	375	325			
BARBITURATES					
Amobarbital	225	217	300	N/A	ng/mL
Butalbital	225	234			
Pentobarbital	225	240			
Phenobarbital	225	245			
Secobarbital	225	240			
BENZODIAZEPINES					
α-Hydroxyalprozolam	225	238	300	N/A	ng/mL
Lorazepam	225	248			
Nordiazepam	225	270			
Oxazepam	225	247			
Temazepam	225	244			
CANNABINOIDS					
(1)-Carboxy-11-nor-Delta-9-THC	11.25	11	15	15	ng/mL
COCAINE					
Benzoylcegonine	112.5	109	150	100	ng/mL
METHADONE					
Methadone	225	211	300	N/A	ng/mL
Methadone Primary Metabolite	225	215	300	N/A	ng/mL
METHAQUALONE	225	215	300	N/A	ng/mL
OPIATES					
6-Monoacetylmorphine	7.5	7	10	10	ng/mL
Codeine	1500	1145	2000	2000	
Morphine-3-β-D-Glucuronide	2430*	1882*	3240*	2000	
	*Equivalent 1500 ng/mL of free morphine	*Equivalent 1162 ng/mL of free morphine	*Equivalent 2000 ng/mL of free morphine	2000	
OPIOIDS					
Hydrocodone	300	259	400	100	ng/mL
Hydromorphone	300	293	400	100	
Oxycodone	225	211	300	100	
Oxymorphone	225	215	300	100	
PHENCYCLIDINE	18.75	19	25	25	ng/mL
PROPOXYPHENE					
Norpropoxyphene	225	194	300	N/A	ng/mL
TRICYCLIC ANTIDEPRESSANTS					
Nortriptyline	225	176	300	N/A	ng/mL

UTAK's express and implied warranties (including merchantability and fitness) are conditioned on the observance of UTAK's insert directions with respect to the use of UTAK's products.

For technical assistance call: UTAK Technical Service (888) 882-5522

UTAK Laboratories, Inc.

25020 Avenue Tibbitts
Valencia, CA 91355
USA
T: (888) 882-5522
F: (661) 294-9272
E: welovecontrol@utak.com

Product Number: 50700

Product Description: DAU HC1

Lot Number: C4043

Expiration Date: 04/22

Volume: 4x10mL Vials, Lyophilized
DFID: LUDOA

EC Authorized Representative
Emergo Europe

Prinsessegracht 20
2514 AP, The Hague
The Netherlands

