

DAU LC 1

URINE TOXICOLOGY CONTROL

I. INTENDED USE:

Many analytes can be measured in patients by using analytical test methods. The UTAK DAU LC 1 Controls are for use as quality control materials for measuring the level of drugs of abuse in urine. All control material is intended for use on a continuous basis so that a statistical evaluation of testing performance can be obtained.

II. SUMMARY AND PRINCIPLES:

Several different techniques are used for evaluating or estimating the variance of results. The three subjects summarized below must be considered with any test method.

1. PREVENTIVE MEASURES:

These measures are usually contained in the design of the test method and include consideration for reagents, equipment, and operator errors. These measures are designed to minimize variance.

2. QUALITY CONTROL MEASURES:

When a quality control sample is analyzed at the same time and in the same manner as a patient specimen, an estimate of variance is obtained for the test method. This estimate of variance can be compared to the acceptable limits of variance of the test method.

3. STATISTICAL ANALYSIS OF PATIENT RESULTS:

As an aid in evaluating overall test results, the past experience of expected results can be compared to the results of any given test run. For example, it would not be expected that all results of a given test run be in an elevated range.

Quality control materials are widely used as a means to aid in the evaluation of test results. The following subjects are to be considered in the use of any control material.

- |                 |                            |
|-----------------|----------------------------|
| 1. Multi-Level  | NORMAL / ELEVATED          |
| 2. Matrix       | HUMAN / ANIMAL / CHEMICAL  |
| 3. Availability | SUFFICIENT FOR STATISTICS  |
| 4. Form         | LIQUID / FROZEN / DRIED    |
| 5. Variety      | DIFFERENT THAN CALIBRATORS |

The UTAK DAU LC 1 Controls are prepared from normal human materials and will generate data that checks and evaluates the results of a test method over the normal and elevated ranges. The principles of statistics require that the same material be available for comparison for any given time period. Dried control materials both extend the usable time period and allow larger quantities to be available. Statistical accuracy requires that a test method be defined for variance and be calibrated with a suitable standard. The quality control materials that are used must be of a sufficient variety so that the measurements and the data that are obtained are independent of the calibration standards. By using a variety of materials, the entire test method can be continuously evaluated to ensure reliable results.

III. PRODUCT DESCRIPTION:

The matrices for the UTAK DAU LC 1 Controls are prepared from normal human urine. Every analyte is added and adjusted to the desired concentration range for each lot prepared (Target Value). Quality control before, during, and after the preparation of the control material ensures that each lot is of the same quality.

IV. PRECAUTIONS:

- Although the urine donors have been tested and found negative for HBsAg by RIA and HIV by EIA, the control material should be treated as any other potentially infectious agent.
- For in vitro diagnostic use only.
- For analytical use only.

V. STORAGE AND STABILITY:

- Store dried control material at 2-8°C (35-46°F). Stable to expiration date printed on the insert and label.
- Store reconstituted control material at 2-8°C (35-46°F). **Stable for 25 days after reconstitution.**

VI. PROCEDURE:

- Remove cap from each vial to be used.
- Reconstitute control material by adding **exactly 10.0 mL of distilled water**, using a **10 mL** volumetric pipette or equivalent.
- Replace cap and let sit 10-15 minutes.
- Swirl gently 3-4 minutes to ensure a homogeneous mixture.
- Swirl gently each time an aliquot is removed to ensure a homogeneous mixture.
- Assay control material in same manner as patient specimens, following the exact same instructions from the entire test method.
- Record the results obtained on a quality control chart that describes statistical limits for the test method and the particular lot of control material.

VII. LIMITATIONS:

- Control material is for use in quality control programs only; it is not intended for use as a calibration standard.
- Check the lot number on each vial to be sure it corresponds to the lot number printed on the insert.
- Results are dependent upon proper storage, reconstitution accuracy, and adequate mixing.
- Control material approximates a patient specimen.

VIII. EXPECTED VALUES:

- Listed in the table below are the Target Value and the Reference Value; the Reference Value is derived from replicate analysis performed by independent laboratory testing.
- The Reference Value is determined by Gas Chromatography (GC).
- Laboratories should establish their own statistical values for precision and expected ranges.

LOW CONFIRMATION

DRUG CLASS Analytes	Units	UTAK Low Cutoffs	DAU LC 1 Product # 50702		SAMHSA Cutoffs
			Lot #: A9254 Exp Date: 09/20		
			Target Value	Reference Value	
<b>AMPHETAMINES</b> <i>d-Amphetamine</i> <i>d-Methamphetamine</i> <i>MDA</i> <i>MDEA</i> <i>MDMA</i>	ng/mL	250	187.5 187.5 187.5 187.5 187.5	185.0 176.5 197.5 181.5 184.0	250
<b>BARBITURATES</b> <i>Amobarbital</i> <i>Butalbital</i> <i>Pentobarbital</i> <i>Phenobarbital</i> <i>Secobarbital</i>	ng/mL	200	150 150 150 150 150	164 155 156 134 155	n/a
<b>BENZODIAZEPINES</b> <i>α-Hydroxyalprazolam</i> <i>Lorazepam</i> <i>Nordiazepam</i> <i>Oxazepam</i> <i>Temazepam</i>	ng/mL	200	150 150 150 150 150	156 160 134 146 135	n/a
<b>CANNABINOIDS</b> <i>(1)-Carboxy-11-nor-Delta-9-THC</i>	ng/mL	15	11.25	12.55	15
<b>COCAINE</b> <i>Benzoylcegonine</i>	ng/mL	100	75	78	100
<b>METHADONE</b> <i>Methadone</i> <i>Methadone Primary Metabolite</i>	ng/mL	200 200	150 150	170 141	n/a
<b>METHAQUALONE</b>	ng/mL	200	150	159	n/a
<b>OPIATES</b> <i>6-Monoacetylmorphine</i> <i>Codeine</i> <i>Morphine-3-β-D-glucuronide</i>	ng/mL	10 300 486*	7.5 225 365*	8.0 217 333*	10 2000 2000
		<small>*Equivalent 300 ng/mL of free morphine</small>	<small>*Equivalent 225 ng/mL of free morphine</small>	<small>*Equivalent 206 ng/mL of free morphine</small>	
<b>OPIOIDS</b> <i>Hydrocodone</i> <i>Hydromorphone</i> <i>Oxycodone</i> <i>Oxymorphone</i>	ng/mL	100 100 50 50	75 75 37.5 37.5	68 78 39.8 41.3	100 100 50 50
<b>PHENCYCLIDINE</b>	ng/mL	25	18.75	20.95	25
<b>PROPOXYPHENE</b> <i>Norpropoxyphene</i>	ng/mL	200	150	159	n/a
<b>TRICYCLIC ANTIDEPRESSANTS</b> <i>Nortriptyline</i>	ng/mL	200	150	146	n/a

UTAK's express and implied warranties (including merchantability and fitness) are conditioned on the observance of UTAK's insert directions with respect to the use of UTAK's products.

For technical assistance call: UTAK Technical Service (800) 235-3442

MANUFACTURER:  
UTAK LABORATORIES, INC.  
25020 AVENUE TIBBITTS  
VALENCIA, CA 91355  
TEL: (661) 294-3935  
FAX: (661) 294-9272  
E-MAIL: INQUIRIES@UTAK.COM

PRODUCT NUMBER:  
50702  
DFID: LUDOA  
4x10ML VIALS, DRIED

EC AUTHORIZED REPRESENTATIVE  
EMERGO EUROPE  
MOLENSTRAAT 15  
2513 BH, THE HAGUE  
THE NETHERLANDS

