

PM PLUS HIGH URINE TOXICOLOGY CONTROL

I. INTENDED USE:

Many analytes can be measured in patients by using analytical test methods. The UTAK PM Plus High Control is for use as a quality control material for measuring the levels of opiates in urine. It is intended for use on a continuous basis so that a statistical evaluation of testing performance can be obtained.

II. SUMMARY AND PRINCIPLES:

Several different techniques are used for evaluating or estimating the variance of results. The three subjects summarized below must be considered with any test method.

1. PREVENTIVE MEASURES:

These measures are usually contained in the design of the test method and include consideration for reagents, equipment, and operator errors. These measures are designed to minimize variance.

2. QUALITY CONTROL MEASURES:

When a quality control sample is analyzed at the same time and in the same manner as a patient specimen, an estimate of variance is obtained for the test method. This estimate of variance can be compared to the acceptable limits of variance of the test method.

3. STATISTICAL ANALYSIS OF PATIENT RESULTS:

As an aid in evaluating overall test results, the past experience of expected results can be compared to the results of any given test run. For example, it would not be expected that all results of a given test run be in an elevated range.

Quality control materials are widely used as a means to aid in the evaluation of test results. The following subjects are to be considered in the use of any control material.

- | | |
|-----------------|----------------------------|
| 1. Multi-Level | NORMAL / ELEVATED |
| 2. Matrix | HUMAN / ANIMAL / CHEMICAL |
| 3. Availability | SUFFICIENT FOR STATISTICS |
| 4. Form | LIQUID / FROZEN / DRIED |
| 5. Variety | DIFFERENT THAN CALIBRATORS |

The UTAK PM Plus High Control is prepared from normal human materials and will generate data that checks and evaluates the results of a test method over the normal and elevated ranges. The principles of statistics require that the same material be available for comparison for any given time period. Dried control materials both extend the usable time period and allow larger quantities to be available. Statistical accuracy requires that a test method be defined for variance and be calibrated with a suitable standard. The quality control materials that are used must be of a sufficient variety so that the measurements and the data that are obtained are independent of the calibration standards. By using a variety of materials, the entire test method can be continuously evaluated to ensure reliable results.

III. PRODUCT DESCRIPTION:

The matrix for the UTAK PM Plus High Control is prepared from normal human urine. Every analyte is added and adjusted to the desired concentration range for each lot prepared (Target Value). Quality control before, during, and after the preparation of the control material ensures that each lot is of the same quality.

IV. PRECAUTIONS:

1. Although the urine donors have been tested and found negative for HBsAg by RIA and HIV by EIA, the control material should be treated as any other potentially infectious agent.
2. For in vitro diagnostic use only.
3. For analytical use only.

V. STORAGE AND STABILITY:

1. Store dried control material at 2-8°C (35-46°F). Stable to expiration date printed on the insert and label.
2. Store reconstituted control material at 2-8°C (35-46°F). **See table for reconstituted stability.**

VI. PROCEDURE:

1. Remove cap from each vial to be used.
2. Reconstitute control material by adding **exactly 5.0 mL of distilled water**, using a **5 mL** volumetric pipette or equivalent.
3. Replace cap and let sit 10-15 minutes.
4. Swirl gently 3-4 minutes to ensure a homogeneous mixture.

5. Swirl gently each time an aliquot is removed to ensure a homogeneous mixture.
6. Assay control material in same manner as patient specimens, following the exact same instructions from the entire test method.
7. Record the results obtained on a quality control chart that describes statistical limits for the test method and the particular lot of control material.

VII. LIMITATIONS:

1. Control material is for use in quality control programs only; it is not intended for use as a calibration standard.
2. Check the lot number on each vial to be sure it corresponds to the lot number printed on the insert.
3. Results are dependent upon proper storage, reconstitution accuracy, and adequate mixing.
4. Control material approximates a patient specimen.

VIII. EXPECTED VALUES:

1. Listed in the table below are the Target Value and Reference Value; the Reference Value is derived from replicate analysis performed by independent reference laboratory testing.
2. The Verified Value is determined by Gas Chromatography / Mass Spectrometry (GC/MS) and High Performance Liquid Chromatography / Tandem Mass Spectrometry (LC-MS/MS).
3. Laboratories should establish their own values for mean and expected ranges; the mean of several determinations may not duplicate the values listed below, but should fall within ±15% of the Target Value.

Lot Number : A8413		Expiration Date : 01/20		
Analyte	Target Value	Reference Value	Units	Reconstituted Stability (days)
Buprenorphine	400	393	ng/mL	25
Norbuprenorphine	400	387	ng/mL	25
Codeine	400	379	ng/mL	25
Norcodeine	400	375	ng/mL	25
Fentanyl	40	42	ng/mL	25
Norfentanyl	40	43	ng/mL	25
Hydrocodone	400	358	ng/mL	25
Norhydrocodone	400	402	ng/mL	25
Hydromorphone	400	384	ng/mL	25
Meperidine	400	397	ng/mL	25
Normeperidine	400	427	ng/mL	25
Methadone	400	379	ng/mL	25
Methadone Primary Metabolite	400	387	ng/mL	25
Morphine	400	392	ng/mL	25
Monoacetylmorphine-6	40	46	ng/mL	25
Oxycodone	400	379	ng/mL	25
Noroxycodone	400	372	ng/mL	25
Oxymorphone	400	394	ng/mL	25
Noroxymorphone	400	370	ng/mL	25
Tapentadol	400	392	ng/mL	25
N-Desmethyltapentadol	400	420	ng/mL	25
Tramadol	400	399	ng/mL	25
O-Desmethyl-cis-tramadol	400	415	ng/mL	25

UTAK's express and implied warranties (including merchantability and fitness) are conditioned on the observance of UTAK's insert directions with respect to the use of UTAK's products.

For technical assistance call: UTAK Technical Service (800) 235-3442

MANUFACTURER:
UTAK LABORATORIES, INC.
 25020 AVENUE TIBBITTS
 VALENCIA, CA 91355
 TEL: (661) 294-3935
 FAX: (661) 294-9272
 E-MAIL: INQUIRIES@UTAK.COM

PRODUCT NUMBER:
91002
5x5ML VIALS, DRIED

EC AUTHORIZED REPRESENTATIVE
 EMERGO EUROPE
 MOLENSTRAAT 15
 2513 BH, THE HAGUE
 THE NETHERLANDS



Rev. 08/16

