

DOA LEVEL 3
BREAST MILK TOXICOLOGY CONTROL

I. INTENDED USE:

Many analytes can be monitored in patients by using analytical test methods. The UTAK DOA Level 3 Control is for use as a quality control material for monitoring the levels of drugs of abuse in breast milk. It is intended for use on a continuous basis so that a statistical evaluation of testing performance can be obtained.

II. SUMMARY AND PRINCIPLES:

Several different techniques are used for evaluating or estimating the variance of results. The three subjects summarized below must be considered with any test method.

1. **PREVENTIVE MEASURES:**
These measures are usually contained in the design of the test method and include consideration for reagents, equipment, and operator errors. These measures are designed to minimize variance.
2. **QUALITY CONTROL MEASURES:**
When a quality control sample is analyzed at the same time and in the same manner as a patient specimen, an estimate of variance is obtained for the test method. This estimate of variance can be compared to the acceptable limits of variance of the test method.
3. **STATISTICAL ANALYSIS OF PATIENT RESULTS:**
As an aid in evaluating overall test results, the past experience of expected results can be compared to the results of any given test run. For example, it would not be expected that all results of a given test run be in an elevated range.

Quality control materials are widely used as a means to aid in the evaluation of test results. The following subjects are to be considered in the use of any control material.

- | | |
|-----------------|----------------------------|
| 1. Multi-Level | NORMAL / ELEVATED |
| 2. Matrix | HUMAN / ANIMAL / CHEMICAL |
| 3. Availability | SUFFICIENT FOR STATISTICS |
| 4. Form | LIQUID / FROZEN / DRIED |
| 5. Variety | DIFFERENT THAN CALIBRATORS |

The UTAK DOA Level 3 Control is prepared from human breast milk and will generate data that checks and evaluates the results of a test method over the normal and elevated ranges. The principles of statistics require that the same material be available for comparison for any given time period. Frozen control materials both extend the usable time and allow larger quantities to be available. Statistical accuracy requires that a test method be defined for variance and be calibrated with a suitable standard. The quality control materials that are used must be of a sufficient variety so that the measurements and the data that are obtained are independent of the calibration standards. By using a variety of materials, the entire test method can be continuously evaluated to ensure reliable results.

III. PRODUCT DESCRIPTION:

The matrix for the UTAK DOA Level 3 Control is prepared from human breast milk. The analytes are added and adjusted to the desired concentration range for each lot prepared (Target Value). Quality control before, during, and after the preparation of the control material ensures that each lot is of the same quality.

IV. PRECAUTIONS:

1. Although the breast milk donors have been tested and found negative for HBsAg by RIA and HIV by EIA, the control material should be treated as any other potentially infectious agent.
2. For in vitro diagnostic use only.
3. For analytical use only.

V. STORAGE AND STABILITY:

1. Store fresh frozen control material at or below -10°C (14°F). Stable to expiration date printed on the insert and label.
2. Store thawed control material at or below 2-8°C (35-46°F). **Thawed stability :14 days.**

VI. PROCEDURE:

1. Allow fresh frozen control material to thaw at room temperature with cap on.
2. Swirl gently 3-4 minutes to ensure a homogeneous mixture.
3. Swirl gently each time an aliquot is removed to ensure a homogeneous mixture

4. Assay control material in same manner as patient specimens, following the exact same instructions from the entire test method.
5. Record the results obtained on a quality control chart that describes statistical limits for the test method and the particular lot of control material.

VII. LIMITATIONS:

1. Control material is for use in quality control programs only; it is not intended for use as a calibration standard.
2. Check the lot number on each vial to be sure it corresponds to the lot number printed on the insert.
3. Results are dependent upon proper storage, reconstitution accuracy, and adequate mixing.
4. Control material approximates a patient specimen.

VIII. EXPECTED VALUES:

1. Listed in the table below are the Target Value and *Reference Value*; the *Reference Value* is derived from replicate analysis performed by independent reference laboratory testing.
2. The Verified Value is determined by Gas Chromatography / Mass Spectrometry (GC/MS) and High Performance Liquid Chromatography / Tandem Mass Spectrometry (LC-MS/MS).
3. Laboratories should establish their own values for mean and expected ranges; the mean of several determinations may not duplicate the values listed below.

Lot Number: C1028		Expiration Date: 12/19		
Analyte	Cutoff	Target Value	<i>Reference Value</i>	Units
Benzoylcegonine	20	40	positive	ng/mL
Cotinine	5	10	10	ng/mL
Caffeine	500	1000	2200	ng/mL
d-Amphetamine	20	40	41	ng/mL
d-Methamphetamine	20	40	39	ng/mL
Ethanol	10	20	20	mg/dL
Methadone	25	50	47	ng/mL
Morphine	20	40	39	ng/mL
Oxazepam	100	200	190	ng/mL
Oxycodone	10	20	20	ng/mL
Phencyclidine	10	20	20	ng/mL
Secobarbital	40	80	positive	ng/mL
(I)-9-Carboxy-11-nor-Delta-9-THC	10	20	19	ng/mL


UTAK's express and implied warranties (including merchantability and fitness) are conditioned on the observance of UTAK's insert directions with respect to the use of UTAK's products.

For technical assistance call: UTAK Technical Service (800) 235-3442

UTAK LABORATORIES, INC.
25020 AVENUE TIBBITTS
VALENCIA, CA 91355
TEL: (661) 294-3935
FAX: (661) 294-9272
E-MAIL: INQUIRIES@UTAK.COM

PRODUCT NUMBER:
21804
5x5mL VIALS, FROZEN

EC AUTHORIZED REPRESENTATIVE
EMERGO EUROPE
MOLENSTRAAT 15
2513 BH, THE HAGUE
THE NETHERLANDS
TEL: +31 (0) 70.345.8570
FAX: +31 (0) 70.346.7299

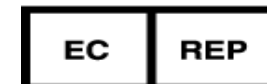

References: CLSI EP05-A2, CLSI EP12-A, CLSI EP14-A2, CLSI EP25-AE, EP17, ISO 13485:2003, ISO 14971:2007