ENGLISH:**I. INTENDED USE:**

The SAMHSA Confirm Level 2, oral fluid toxicology control is for use as a control material to evaluate the performance of an analytical test method.

II. SUMMARY AND PRINCIPLES:

Several different techniques are used for evaluating or estimating the variance of results. The three subjects summarized below must be considered with any test method.

1. PREVENTIVE MEASURES:

These measures are usually contained in the design of the test method and include consideration for reagents, equipment, and operator errors. These measures are designed to minimize variance.

2. CONTROL MEASURES:

When a control sample is analyzed at the same time and in the same manner as a patient specimen, an estimate of variance is obtained for the test method. This estimate of variance can be compared to the acceptable limits of variance of the test method.

3. STATISTICAL ANALYSIS OF PATIENT RESULTS:

As an aid in evaluating overall test results, the past experience of expected results can be compared to the results of any given test run. For example, it would not be expected that all results of a given test run be in an elevated range.

Control materials are widely used as a means to aid in the evaluation of test results. The following subjects are to be considered in the use of any control material.

1. Multi-Level	NORMAL / ELEVATED
2. Matrix	HUMAN / ANIMAL / CHEMICAL
3. Availability	SUFFICIENT FOR STATISTICS
4. Form	LIQUID / FROZEN / DRIED
5. Variety	DIFFERENT THAN CALIBRATORS

III. PRECAUTIONS:**1. For *in vitro* diagnostic use only.****IV. STORAGE AND STABILITY:**

1. Store fresh frozen control material at or below -10°C (14°F). Stable to expiration date printed on the insert and label.
2. Store thawed control material at or below 2-8°C (35-46°F). See table for thawed stability.

V. PROCEDURE:

1. Allow fresh frozen control material to thaw at room temperature with cap on.
2. Swirl gently 3-4 minutes to ensure a homogeneous mixture.
3. Swirl gently each time an aliquot is removed to ensure a homogeneous mixture.
4. Assay control material in same manner as patient specimens, following the exact same instructions from the entire test method.
5. Record the results obtained on a control chart that describes statistical limits for the test method and the particular lot of control material.

VI. LIMITATIONS:

1. Results are dependent upon proper storage and adequate mixing.
2. Control material approximates a patient specimen.

VII. EXPECTED VALUES:

1. Laboratories should establish their own mean values; an individual laboratory's mean of several determinations may not duplicate the values listed below.

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REF

81821

Product Description: SAMHSA Confirm Level 2

Product Number:

LOT

D2235

2023-11-30

Lot Number:

Expiration Date:

Analyte	Thawed Stability (Days)	Gravimetrically Weighed Value	Verified Value	Units
Alprazolam	25	62.5	66.7	ng/mL
Amobarbital	25	150	144.3	ng/mL
d-Amphetamine	25	18.75	21.5	ng/mL
d-Methamphetamine	25	18.75	18.4	ng/mL
(±)-3,4-Methylenedioxymethamphetamine (MDMA)	25	18.75	19.1	ng/mL
Benzoylecgonine	25	10	14.1	ng/mL
Buprenorphine	14	10	9.9	ng/mL
Butalbital	25	150	138.5	ng/mL
Carisoprodol	25	150	124.2	ng/mL
Cocaine	25	10	8.5	ng/mL
Codeine	25	18.75	21.5	ng/mL
Diazepam	25	62.5	69.6	ng/mL
Ethanol	25	125	Present*	mg/dL
Fentanyl	25	10	10	ng/mL
Hydrocodone	25	18.75	21.9	ng/mL
Hydromorphone	25	18.75	19.6	ng/mL
Lorazepam	25	62.5	62.9	ng/mL
Methadone	25	62.5	65.9	ng/mL
6-Monoacetylmorphine (6-MAM)	25	2.5	2.7	ng/mL
Morphine	25	18.75	19.7	ng/mL
Oxazepam	25	62.5	67.4	ng/mL
Oxycodone	25	18.75	19.5	ng/mL
Oxymorphone	25	18.75	20.9	ng/mL
Pentobarbital	25	150	130.6	ng/mL
Phencyclidine (PCP)	25	2.5	2.7	ng/mL
Phenobarbital	25	150	147.5	ng/mL
Propoxyphene	25	62.5	42	ng/mL
Secobarbital	25	150	136.3	ng/mL
Tapentadol	25	62.5	51.4	ng/mL
Temazepam	25	62.5	67.5	ng/mL
Tramadol	25	62.5	63.5	ng/mL
Zolpidem	25	62.5	40.6	ng/mL
(-)-Delta-9-THC	14	2.5	1.8	ng/mL
Matrix: Oral Fluid				
Product Form: Frozen				
Storage: -10°C (14°F)				
Vial Volume: 3 mL				

*Analyte has been weighed and spiked into this control material; however, no claim will be made for verified value.

UTAK's express and implied warranties (including merchantability and fitness) are conditioned on the observance of UTAK's insert directions with respect to the use of UTAK's products.

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For technical support please call: UTAK Technical Service (888) 882-5522

IVD

UTAK Laboratories, Inc.
25020 Avenue Tibbitts
Valencia, CA 91355
T: (888) 882-5522
F: (661) 294-9272
E: welovecontrol@utak.com

EC REP

CE

EMERGO EUROPE
PRINSESSEGRACHT 20
2514 AP THE HAGUE
THE NETHERLANDS